



## DNA EXTRACTION PROCEDURE

- 01 Complete Requisition Form
- 02 Review Information and have Patient Sign Consent on Requisition Form
- 03 Before Submitting Sample Make Sure All of The Requisition Form is Complete and Correct
- 04 Make Sure A Copy of Demographics and Insurance Information is Included
- 05 Provide two sided buccal Swabs to the patient
- 06 Roughly rinse out the subject's mouth twice with water
- 07 Collect tissue by rolling the Catch-All sample collection swab firmly on the inside of the cheek, approximately 20 times on each side (45 seconds on each side), making certain to move the brush over the entire cheek.
- 08 Place the swab end of the Catch-All sample collection swab into a tube container. Seal the tube with the cap and write patient name and DOB on the tube
- 09 Then Place The Sample in the Biohazard Bag With All Completed Paperwork
- 10 Place the Sealed Biohazard Bag into the Shipping Box
- 11 Send the Box on a Scheduled Pick-up
- 12 Log The Patient DNA Sample for the Doctors Files