



## UDS COLLECTION PROCEDURE

- 01 Inform Patient Test is for Medication Monitoring
- 02 Complete Requisition Form
- 03 Get Current Medication List From Patient
- 04 Review Information and have Patient Sign Consent on Requisition Form
- 05 Print Patient Name, Date of Collection and Date of Birth with a Sharpie on the Top of the Collection Cup
- 06 Instruct Patients That They Must Provide 30cc of Urine or Exceed the Min Volume Line
- 07 Before Submitting Sample Make Sure All of The Requisition Form is Complete and Correct
- 08 Make Sure A Copy of Demographics and Insurance Information is Included
- 09 Include List Of Patient's Current Medications
- 10 Then Place The Sample in the Biohazard Bag With All Completed Paperwork
- 11 Ensure That The Specimin is Placed in a Separate Compartment of The Bag From The Paperwork
- 12 Send the Box on a Scheduled Pick-up
- 13 Log The Patient UDS Sample for The Doctors Files